Glide

2 in 1 Rollator and Transport Chair

Uplivin UK Ltd.

Suite 12/3D, Docklands Business Centre
10-16 Tiller Rd, London, E148PX, UK

info@uplivin.com
www.uplivin.com



MedNet EC-REP GmbH
Borkstrasse 10, 48163 Muenster, Germany

Jan/2025 V1.0


English
Deutsch
Español
Français
Italiano
Dutch
Svenska

uplivin®
Uplivin UK Ltd.

WARNING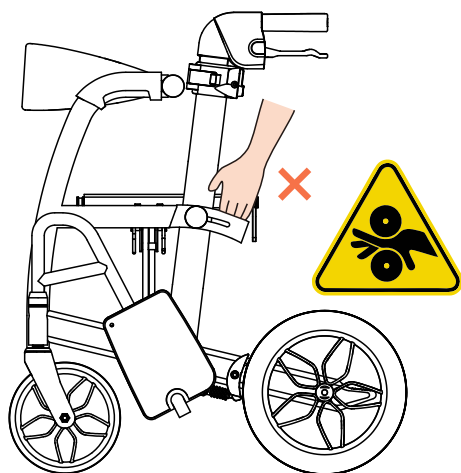
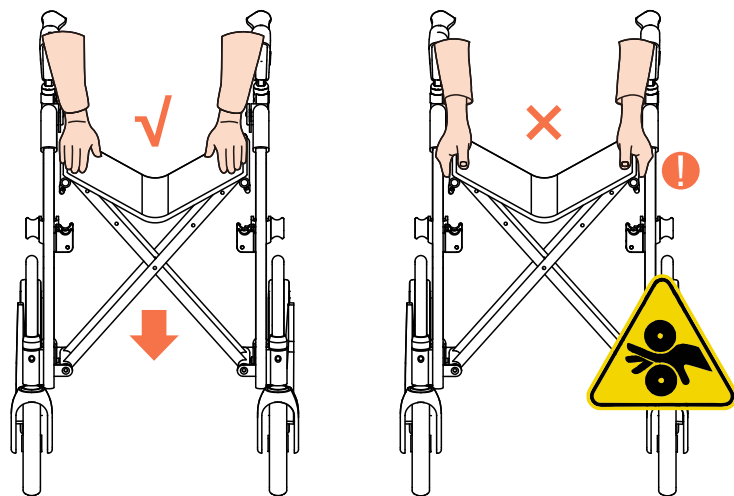


A

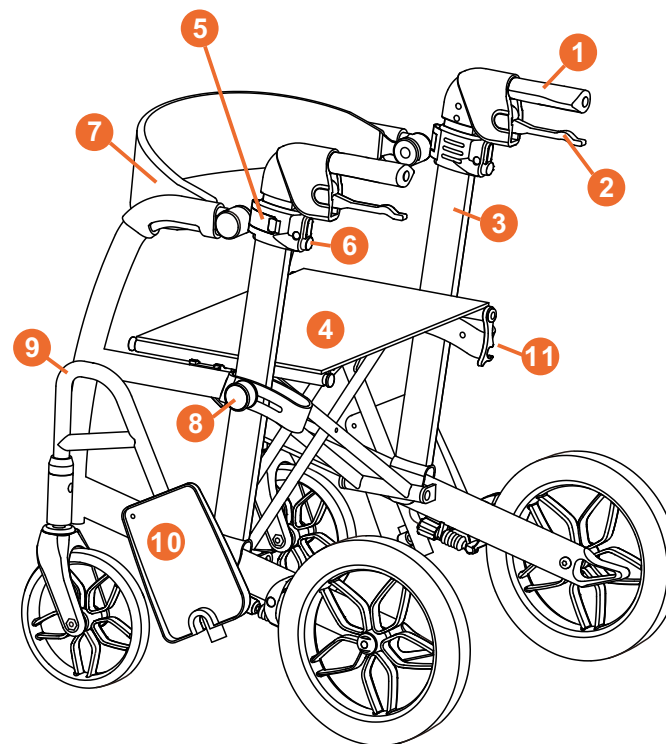
B


C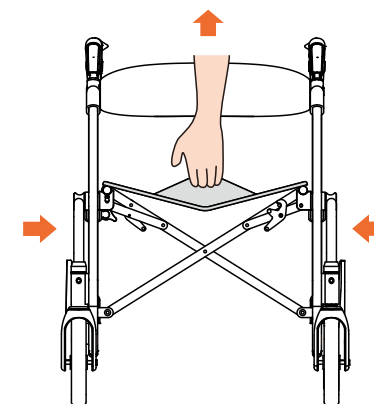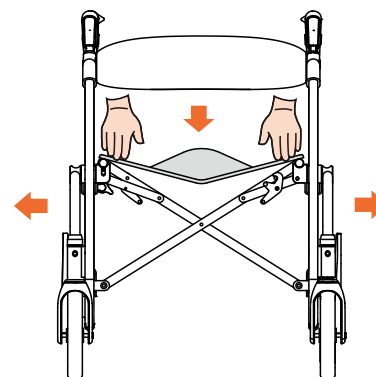


D

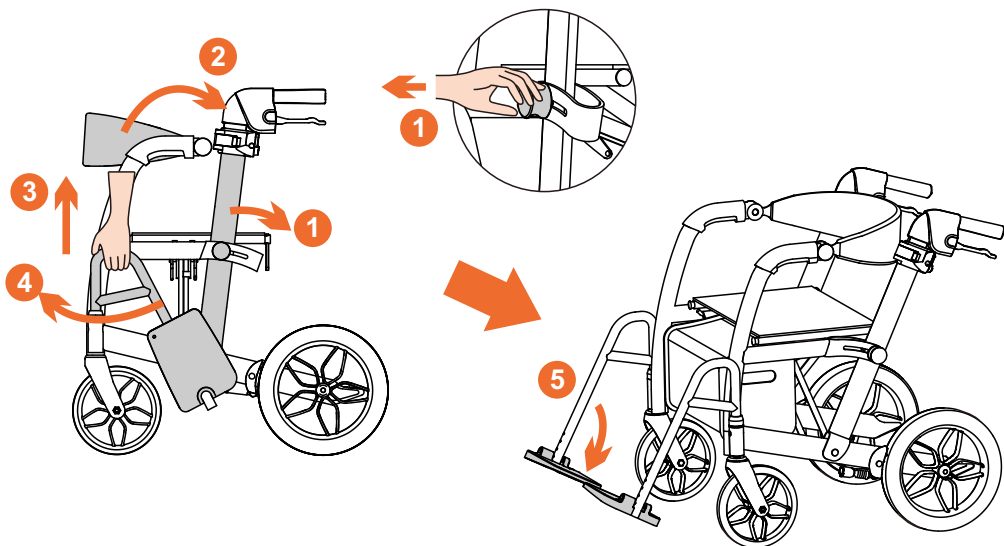
E

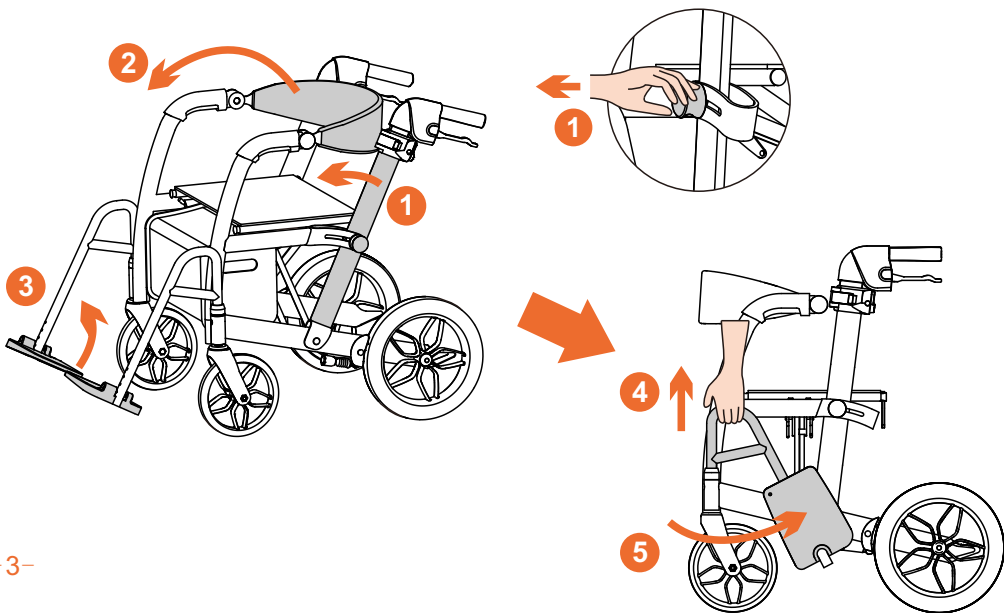
F

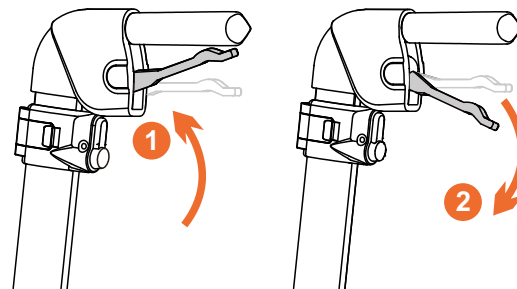
G

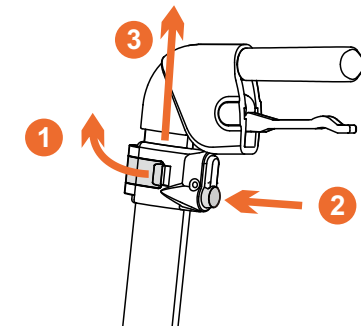
H

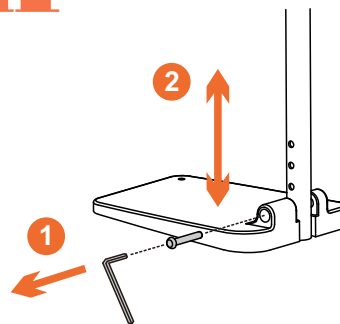
I


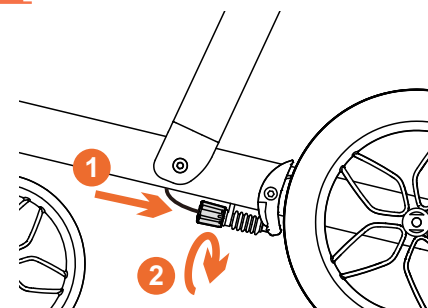
J

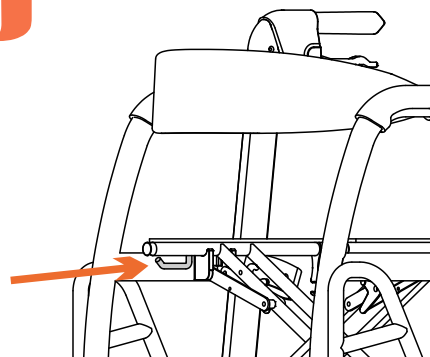K

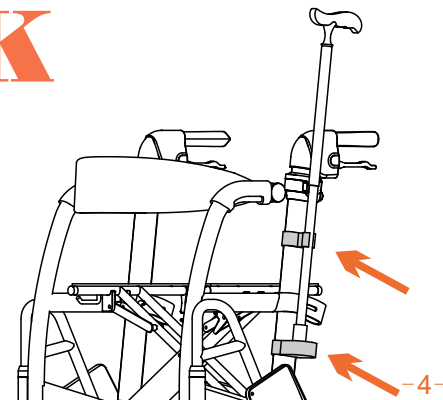Glide

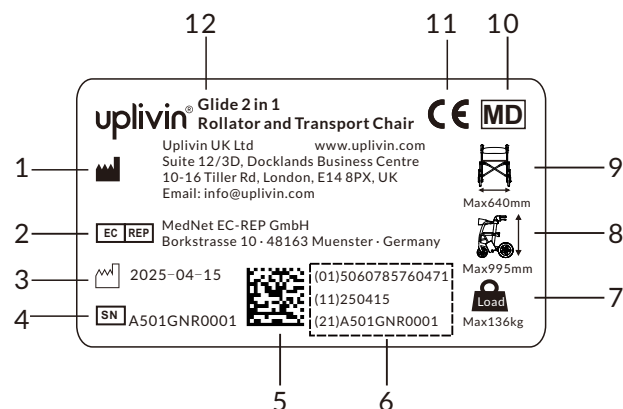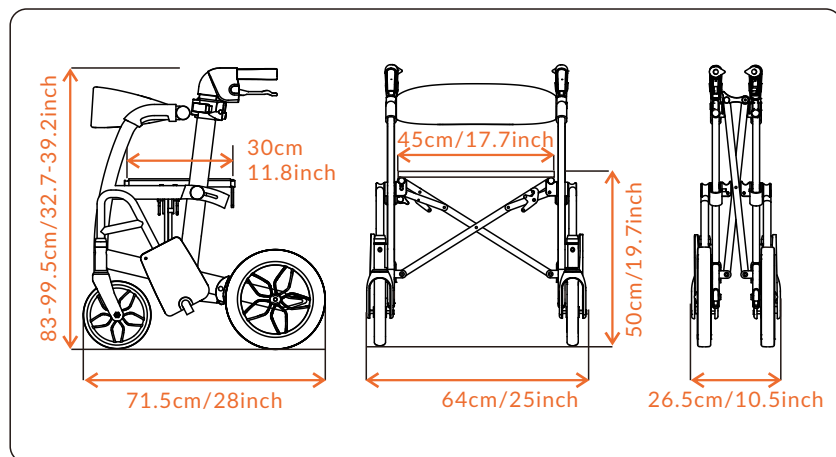- | | |
|--------------------------------------|-----------------------|
| 1 Manufacturer | 7 Maximum user weight |
| 2 European Authorised Representative | 8 Maximum height |
| 3 Date of manufacture | 9 Maximum width |
| 4 Serial number | 10 Medical device |
| 5 Gs1 DataMatrix | 11 CE label |
| 6 Unique Device Identifier | 12 Name of product |

English	User Manual.....	7
Deutsch	Bedienungsanleitung.....	14
Español	Manual de usuario.....	22
Français	Manuel d'utilisation.....	29
Italiano	Manuale d'uso.....	37
Dutch	Gebruiksaanwijzing.....	44
Svenska	Användarmanual.....	51

Congratulations on purchasing the Glide 2-in-1 Rollator and Transport Chair. It is designed to be used as both a rollator and a transport chair for those with poor physical strength or limited mobility. It can be used both indoors and outdoors.

This product will improve your mobility to a better level than traditional rollators, and make everyday life easier, safer, and more enjoyable. The product as a transport chair enables the user to sit comfortably while being pushed.

Please ensure you read these instructions thoroughly before using the product.

Warnings:

- Before using the Glide, check that it is fully unfolded and fixed into position and that the brakes are functioning properly.
- Both brake handles must be applied at the same time whether the driving brake or the parking brake is being used.
- Make sure all the components are locked in position before use. Do not use the product if any part has been damaged, broken or cracked, or is missing.
- Before you sit on the seat of the Glide when used as a rollator, the parking brakes must be applied using both handles.
- The Glide should not be pushed forward if the parking brakes or the driving brakes have been applied.
- The Glide may only be used to seat one person under 136 kg in weight. It should not be used to carry heavy goods.
- Do not try to adjust the push tubes forward or backward when it is being used as a seat.
- The user should not use the Glide as a transport chair when travelling in a taxi, bus or car. It is not suitable for this type of use.

- The Glide must not be used when climbing stairs or used on rough terrain such as gravel, grass, sand, and soil.
- Do not lean forward when sitting on the Glide as a transport chair as it may tip over.
- When converting the Glide between rollator and transport chair mode, or when folding and unfolding the unit, care should be taken not to trap your fingers in the frame.
- Children should be warned that the Glide is not a toy.
- When the Glide is exposed to extreme temperatures, care should be taken as certain parts may become very hot or very cold.
- Do not wear long items of clothing when using the Glide to avoid material becoming caught in the wheels resulting in injury to the user.
- After the Glide is converted into a transport chair, it should not be used as a rollator.

A Part Description

- ① Push handles
- ② Brake levers
- ③ Push tubes
- ④ Seat
- ⑤ Handle-tightening levers
- ⑥ Height-adjustment buttons
- ⑦ Backrest
- ⑧ Lock pins
- ⑨ Footrest frames
- ⑩ Footrests
- ⑪ Hook

B Unfolding the Glide

- ①Unlock the hook at the back end of the seat.
- ②Press the two sides of the seat down with flattened palms to easily unfold the Glide. A click sound indicates that the unit has been unfolded properly.

Warning: Do not put fingers under the seat when pressing down to avoid pinching!

C Folding the Glide

- ①Pull up the strap in the middle of the seat to fold the Glide.
- ②Lock the hook at the back ends of the seat after it has been folded properly.

D How to Turn the Glide from a Rollator to a Transport Chair

Apply the parking brakes on both sides before continuing.

- ①Pull the two push tubes backwards while pulling up the lock pins until a click is heard. Be careful not to catch your fingers in the gap between the tube and the frame.

- ②Flip the backrest to the rear.
- ③Pull up the frame of the footrests.
- ④Flip the footrests to the front.
- ⑤Set the footrests down.

Release the parking brakes when completed.

E How to Turn the Glide from a Transport Chair to a Rollator

Apply the parking brakes on both sides before proceeding.

- ①Pull the two push tubes forward while pulling up the lock pins until a click is heard. Be careful not to catch your fingers in the gap between the tube and the frame.
- ②Flip the backrest to the front.
- ③Fold up the footrests.
- ④Pull the footrest frame up.
- ⑤Flip the footrests to the sides.

Release the parking brakes when completed.

F How to use the driving brakes and parking brakes

- ① Pulling the brake levers up will slow the Glide down until it stops, acting as driving brakes.
- ② Pushing the brake levers down until it stops will act as parking brakes.

G How to Adjust the Height of the Push Handles

For health and safety purposes, it is important to adjust the handle height to the correct height for the user. To achieve this, the user should stand between the handles of the Glide rollator with their arms loosely by their sides, elbows bent naturally and with the palms of both hands turned inwards. The position of the wrists indicates the correct height of the handle for the user.

Adjustment Steps:

- ① Loosen the two handle-tightening levers.
- ② ③ While pressing the height-adjustment button down, move the push handle up or down on both sides.
- ④ Fasten the two handle-tightening levers.

*Please note that to achieve the correct posture and evenly distributed body weight, both handles must be adjusted to the same height.

H How to adjust the height of footrests

- ① Loosen the footrest screw with a 5mm Allen key and remove the footrest.
- ② Select the appropriate hole on the footrest frame to achieve the desired height, align the footrest, insert the screw and tighten. Repeat for the other side.

I How to adjust the brakes

The brake adjustment screw is located near the rear wheel brake point. Push the head of the screw towards the rear wheel and rotate it to adjust the brakes if they are too tight or too loose.

J Storage Bag

Storage bag mounted on two forks under the seat.

K Walking stick holder

Walking stick holder mounted on side of one of the push tubes.

Maintenance

The Glide should be cleaned using a general-purpose detergent and tepid water. It should not be stored in a damp or very hot environment.

Warranty

Our two-year warranty does not cover normal wear and tear of certain parts such as the wheels and cables.

Specifications

Overall width	25.1" / 64cm
Folded width	10.6"/26.5cm
Overall length	28.1"/71.5cm
Handle height range	32.6"-39.1"/83cm-99.5cm
Seat height	19.6"/50cm
Seat size	17.7"x 11.8"/45cm x 30cm
Weight	24 lbs / 10.9kg
Weight capacity	300 lbs/ 136kg
Wheel diameter	Front 8", Rear 12"
Storage bag capacity	11 lbs/5kg

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